


**INFORMATIVE NOTE**
**Name: REFLEXX 77**
**Date: 01/01/2024 Rev: 08**
**MANUFACTURER**

 REFLEXX SpA  
 Via Passeri, 2 46019 VIADANA (MN)  
 Tel.: + 39 0375 758891 Fax + 39 0375 464504 [info@reflexx.com](mailto:info@reflexx.com)  
[www.reflexx.com](http://www.reflexx.com)


Cod./Size	Pack	
	Box	Carton
R77100F/S	8032891633878	8032891638873
R77100F/M	8032891633892	8032891638897
R77100F/L	8032891633922	8032891638927
R77100F/XL	8032891633953	8032891638958
R77100/XS	8032891633465	8032891638460
R77100/S	8032891633472	8032891638477
R77100/M	8032891633489	8032891638484
R77100/L	8032891633496	8032891638491
R77100/XL	8032891633502	8032891638507
R77/XS	8032891633809	8032891638804
R77/S	8032891633823	8032891638828
R77/M	8032891633847	8032891638842
R77/L	8032891633861	8032891638866
R77/XL	8032891633885	8032891638880

**DESCRIPTION:** ■ disposable glove powder free, not sterile ■ **Material:** nitrile ■ **Shape:** ambidextrous; five fingers, beaded cuff ■ **Colour:** light blue ■ **Feature:** internally chlorinated; textured surface and textured fingertips ■ **Packing:** 100 or 200 pieces/box; 10 boxes/carton ■ **ISO 2859:** AQL 1.5 (level G1) freedom from holes; AQL 2.5 (level G1) major defects; AQL 4 (level G1) minor defects

**CONFORMITY AND PERFORMANCES**
**Personal Protective Equipment cat III**

 0465 ANCI CIMAC, Via Aguzzafame, 60/b 27029 Vigevano (PV)  
 Reg EU 2016/425 - EN ISO 374-1:2016 TYPE C - EN ISO 374-5:2016 VIRUS (ISO 16604:2004). - EN 374-2:2014 - EN 420:2003 + A1 2009

**Medical Device**

 Reg EU 2017/745  
 (CND T01020204 NON SURGICAL NITRILE GLOVES)  
 - Complies with ASTM-F-1671 (viral penetration test) and ISO 10993-10 regulations (skin irritation and sensitization)


Reg CE 1935/2004 – Reg EU 10/2011 - DM 21/03/73. To such use carefully read the declaration of suitability for food contact (DICA).

**DIMENSIONS**

Cod./Size		Length min* (mm)	Width (±5mm)
/	R77100/ XS (6)	R77/ XS (6)	240
R77100F/S (7)	R77100/ S (7)	R77/ S (7)	240
R77100F/M (8)	R77100/ M (8)	R77/ M (8)	240
R77100F/L (9)	R77100/ L (9)	R77/ L (9)	240
R77100F/XL (10)	R77100/ XL (10)	R77/ XL (10)	240

\* Gloves for special applications 5.1.3

<b>Thickness mm</b>	Palm (+/- 0.02)	0.06
---------------------	-----------------	------

<b>EN 455-2:2009/A1</b>	Tensile strength	≥6 N
-------------------------	------------------	------

<b>EN ISO 374-5:2016</b>	Protection from microorganisms (bacteria and fungi)	Virus protection ISO 16604:2004
	pass	pass

EN ISO 374-1:2016				
Chemical compound :				
COD	EN 16523-1:2015 TYPE C		EN 374-4:2013	
	Performance level	Breakthrough time	% degradation	
K	3	>60'	7,1%	

**REG EU 2016/425**

 EN ISO 374-1:2016 +  
A1:2018/Type C

**EN ISO 374-5: 2016**

**RISKS FROM WHICH THE DPI IS DESIGNED TO PROTECT:**

Chemical compound :	Protection
K Sodium hydroxide 40 % CAS:1310-73-2	Protection from use of inorganic bases

**RECOMENDATIONS OF USE** Multi-purpose glove to protect the hand and/or the handled product, can also be used for general uses ■ The glove is suitable for contact with pure chemical products for a period of less than the permeation time indicated in the chemical resistance table ■ Results obtained under laboratory conditions may not reflect actual working time **WARNINGS** Before using the product, try it in the real condition of use ■ The glove is a device subject to wear, so frequent replacement is recommended and in all cases where imperfections are observed ■ Wear gloves with dry, clean hands ■ In rare cases where transient hypersensitive skin reactions may occur, discontinue use and consult a doctor ■ The user must inform the manufacturer and the relative authority of any incidents occurring during the use of the protective equipment **STORAGE AND DISPOSAL** Store the gloves in their original packaging in a dry place. ■ Do not expose to direct sunlight and heat sources. ■ The expiry date is 3 years from the date of manufacture indicated on the sales package, and is valid for the product properly stored ■ Gloves must be disposed of according to local regulations ■ Product Declaration of Conformity on [www.reflexx.com](http://www.reflexx.com)