Reflexx S.p.A., Via Passeri 2, 46019 Viadana (MN) | ITALY DISPOSABLE EXAMINATION NITRILE GLOVE

> Reflexx N100 USER INFORMATION SHEET

rev. UIS 01 june 2024

(€ 2797

PPE cat. III. Peg. EU 2016/425 EN ISO 374-1/2016+41/2018 TVPE 4 EN ISO 374-5:2016 VIRUS (ISO 16604:2004) EN ISO 21420:2020 and ISO 21420:2020/Amd.1:2022

Chamical compound :		EN ISO 374-1:2018 (A1:2018 TYPE A		EN 374-4(2019
	CCD	Performance level	Breakthrough time	% degradation
n-Heplane		8	×480'	13.2
Sodium Hydraxide 40%	к	8	×430'	-1.2
Ammonium Hydroxide 25%	0	5	>240'	29.0
Hydrogen Percoide 30%	P	6	×430′	7.3
Formaldehyde 37%	т	6	×480′	-12.2
Sulphuric Acid 98%	L	2	×30'	103.0
Nethanol	A	1	>10'	76.7
Isapropyl Alashol		6	>430'	-31.9
Hydrofluoric Apid 40%	S	2	>30'	64,9

Module D on going conformity by 2797 BSI Group The Netherlands B.V. - Say EU type examination (Module B) by 2797 BSI Group Netherlands B.V. - Say Building, John M. Keynesplein 9, 1066 EP Amsterdam, Netherlands OTHER COMPLIANCE

Medical Device CLASS I | MDR (EU) 2017/745 - EN 455 1,2,3 & 4

Reg. CE 1935/2004 - Reg. UE 10/2011 - DM 21/03/1973 FOOD CONTACT

EN 374-4:2019 Degradation results indicate the change in puncture resistance of the gloves after exposure to the challenge chemical

EU Declaration of Conformity on www.reflexx.com

 Disposable examination nitrile glove

 Powder free, non-sterile
 Shape: ambidextrous: five fingers
 Sizes: 6-10
 Please read this USER INFORMATION

 sheet carefully before using the glove.

RECOMMENDATIONS OF USE

Multi-purpose glove to protect the hand and / or the handled product = Glove suitable for contact with pure chemical products for less than the permeation time indicated in the chemical resistance table The results obtained under laboratory conditions may not reflect the service life when used.

WARNINGS

 This information does not reflect the actual duration of protection in the workplace and the differentiation between mixtures and pure chemicals. The chemical resistance has been assessed under laboratory conditions from samples taken from the palm only (except in cases where the glove is equal to or over 400 mm – where the cuff is tested also) and relates only to the specimen tested. It can be different if the chemical is used in a mixture

The penetration resistance have been assessed under laboratory conditions and relates only to the tested specimens . It is recommended to check that the gloves are suitable for the intended use because the conditions at the workplace may differ from the type test depending on temperature, abrasion and degradation **.** When used protective gloves may provide less resistance to the dangerous chemical due to changes in physical properties. Movements, snagging, rubbing, degradation caused by the chemical contact etc. may reduce the actual use time significantly. For corrosive chemicals, degradation can be the most important factor to consider in selection of chemical resistant gloves . Before usage, inspect the gloves for any defect or imperfections . Before using the product, try it in real conditions . Device subject to wear frequent replacement is recommended and in all cases where imperfections are seen
Wear gloves with clean dry hands
In rare cases where transient hypersensitive skin reactions may occur discontinue use and consult a doctor . The user must inform the manufacturer and the relative authority of any incidents occurring during the use of the protective equipment
Donning – Hold glove by the bead with one hand. Align the glove thumb with your other hand thumb and slide your hand into the glove, one finger into each glove finger. Pull by the glove palm to get a good fit. Don the other glove by the same procedure

Doffing – Hold glove bead and pull toward the finger until the

Leaflet: paper PAP22 Recycling bin. Always check the disposal rules of your municipality

glove come off STORAGE AND DISPOSAL

Store at room temperature prevailing in respective countries The expiry date is 3 years from the date of manufacture indicated on the sales package, and is valid for the product properly stored . Gloves are packed in dispenser which is suitable for transports. Keep the gloves in the box when not in use

Gloves must be disposed of according to local

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