


INFORMATIVE NOTE

Name: REFLEXX N80B		Date: 20/03/2024 Rev: 03		
MANUFACTURER REFLEXX SpA Via Passeri, 2 46019 VIADANA (MN) Tel.: + 39 0375 758891 Fax + 39 0375 464504 info@reflexx.com www.reflexx.com			Pack	
			Cod./Size	Box
	N80B/XS	8032891630181	8032891635186	
	N80B/S	8032891631065	8032891636060	
	N80B /M	8032891631072	8032891636077	
	N80B /L	8032891631089	8032891636084	
	N80B/XL	8032891631096	8032891636091	
	N80B200/XS	8032891630198	8032891635193	
	N80B200/S	8032891630143	8032891635148	
	N80B200/M	8032891630150	8032891635155	
	N80B200/L	8032891630167	8032891635162	
	N80B200/XL	8032891630174	8032891635179	

DESCRIPTION: ■ disposable glove powder free, not sterile ■ **Material:** nitrile ■ **Shape:** ambidextrous; five fingers, beaded cuff ■ **Colour:** blue ■ **Feature:** internally chlorinated; textured surface and textured fingertips ■ **Packing:** 100 or 200 pieces/box; 10 boxes/carton ■ **ISO 2859:** AQL 1.5 (level G1) freedom from holes; AQL 2.5 (level G1) major defects; AQL 4 (level G1) minor defects

CONFORMITY AND PERFORMANCES
Personal Protective Equipment cat III

0465 ANCI_CIMAC, Via Aguzzafame, 60/b 27029 Vigevano (PV)

Reg EU 2016/425 - EN ISO 374-1:2016 TYPE C - EN ISO 374-5:2016 - EN 374-2:2014 - EN 420:2003 + A1 2009

Medical Device

Reg EU 2017/745 – EN 455 1,2,3 & 4

(CND T01020204 NON SURGICAL NITRILE GLOVES)



cat III

classe I



Reg CE 1935/2004 – Reg EU 10/2011 - DM 21/03/73. To such use carefully read the declaration of suitability for food contact (DICA).



DIMENSIONS			
Cod./Size		Length min(mm)	Width (±5 mm)
N80B/XS (6)	N80B200/XS (6)	240	75
N80B/S (7)	N80B200/S (7)	240	85
N80B/M (8)	N80B200/M (8)	240	95
N80B/L (9)	N80B200/L (9)	240	106
N80B/XL (10)	N80B200/XL (10)	240	116

Thickness mm	Palm (+/- 0.02)	0.06
---------------------	-----------------	------

EN 455-2:2009/A1	Tensile strength	≥6 N
-------------------------	------------------	------

EN ISO 374-5:2016	Protection from microorganisms (bacteria and fungi)	Virus protection ISO 16604:2004
	pass	pass

EN ISO 374-1:2016				
Chemical compound :	COD	EN 16523-1:2015 TYPE C		EN 374-4:2013
		Performance level	Breakthrough time	% degradation
Sodium hydroxide 40%	K	6	>480'	6,5%

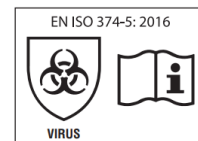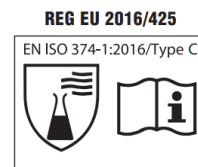

EN 420:2003 + A1:2009

RISKS FROM WHICH THE DPI IS DESIGNED TO PROTECT:

Chemical compound :	Protection
K Sodium hydroxide 40 %	CAS:1310-73-2 Protection from use of inorganic bases

RECOMENDATIONS OF USE Multi-purpose glove to protect the hand and/or the handled product, can also be used for general uses ■ The glove is suitable for contact with pure chemical products for a period of less than the permeation time indicated in the chemical resistance table ■ Results obtained under laboratory conditions may not reflect actual working time **WARNINGS** Before using the product, try it in the real condition of use ■ The glove is a device subject to wear, so frequent replacement is recommended and in all cases where imperfections are observed ■ Wear gloves with dry, clean hands ■ In rare cases where transient hypersensitive skin reactions may occur, discontinue use and consult a doctor ■ The user must inform the manufacturer and the relative authority of any incidents occurring during the use of the protective equipment

STORAGE AND DISPOSAL Store the gloves in their original packaging in a dry place. ■ Do not expose to direct sunlight and heat sources. ■ The expiry date is 3 years from the date of manufacture indicated on the sales package, and is valid for the product properly stored ■ Gloves must be disposed of according to local regulations ■ Product Declaration of Conformity on www.reflexx.com