


**INFORMATIVE NOTE**

<b>Name: REFLEXX N82</b>		<b>Date: 15/05/2024 Rev: 01</b>	
<b>MANUFACTURER</b> REFLEXX SpA Via Passeri, 2 46019 VIADANA (MN) Tel.: + 39 0375 758891 Fax + 39 0375 464504 <a href="mailto:info@reflexx.com">info@reflexx.com</a> <a href="http://www.reflexx.com">www.reflexx.com</a>			
		<b>Pack</b>	
<b>Cod./Size</b>	<b>Box</b>	<b>Carton</b>	
<b>N82/XS</b>	8032891630341	8032891635346	
<b>N82/S</b>	8032891630358	8032891635353	
<b>N82/M</b>	8032891630365	8032891635360	
<b>N82/L</b>	8032891630372	8032891635377	

**DESCRIPTION:** ■ disposable examination glove powder free, not sterile ■ **Material:** nitrile ■ **Shape:** ambidextrous; five fingers, beaded cuff ■ **Colour:** pink ■ **Feature:** internally chlorinated; externally textured fingertips ■ **Packing:** 100 pieces/box; 10 boxes/carton ■ **ISO 2859:** AQL 1.5 (level G1) freedom from holes; AQL 2.5 (level G1) major defects; AQL 4 (level G1) minor defects

**CONFORMITY AND PERFORMANCES**
**Medical Device**

Reg EU 2017/745 – EN 455 1,2,3 & 4  
 (CND T01020204 NON SURGICAL NITRILE GLOVES)

class I

**Personal Protective Equipment cat I** for minimal risks only  
 Reg EU 2016/425 - EN 420:2003 + A1 2009

cat I

DIMENSIONS		
Cod./Size	Length (mm)	Width mm (±5)
N82/XS (6)	240	80
N82/S (7)	240	85
N82/M (8)	240	95
N82/L (9)	240	105

<b>Thickness</b> mm	Palm (+/- 0.02)	0.06
<b>EN 455-2:2009/A1</b>	Tensile strength	≥ 6N

**RECOMENDATIONS OF USE** Multi-purpose glove to protect the hand and/or the handled product ■ Avoid contact with aggressive chemical elements and actions that could tear the glove **WARNINGS** Before using the product, try it in the real condition of use ■ The glove is a device subject to wear, so frequent replacement is recommended and in all cases where imperfections are observed ■ Wear gloves with dry, clean hands ■ In rare cases where transient hypersensitive skin reactions may occur, discontinue use and consult a doctor ■ The user must inform the manufacturer and the relative authority of any incidents occurring during the use of the protective equipment **STORAGE AND DISPOSAL** Store the gloves in their original packaging in a dry place ■ Store at room temperature ■ Do not expose to direct sunlight and heat sources. ■ The expiry date is 3 years from the date of manufacture indicated on the sales package, and is valid for the product properly stored ■ Gloves must be disposed of according to local regulations ■ Product Declaration of Conformity on [www.reflexx.com](http://www.reflexx.com)